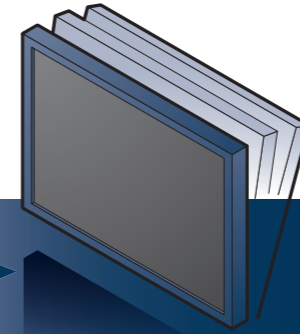
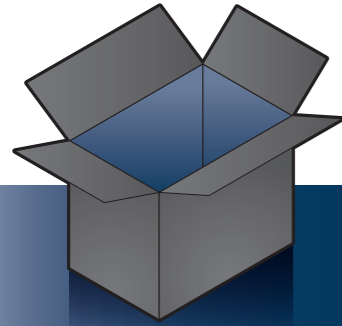
# RadiForce® MS235WT

Touch Panel Color LCD Monitor

Instructions for Use **Gebrauchsanweisung** **Notice d'instruction** **使用说明书**  
**Setup Guide** **Installationsanleitung** **Guide d'installation** **设定指南**

**Important** : Please read PRECAUTIONS, this Setup Guide and the User's Manual stored on the CD-ROM carefully to familiarize yourself with safe and effective usage. Please retain this guide for future reference.

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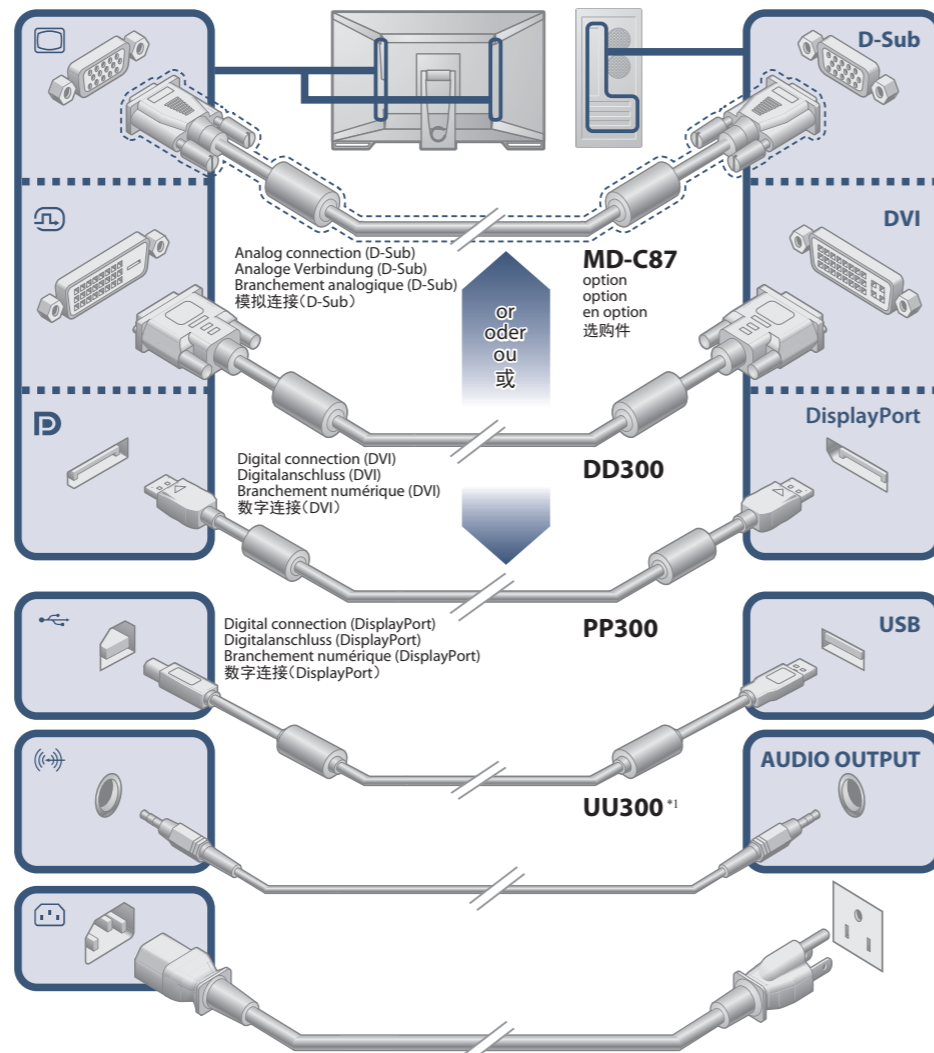
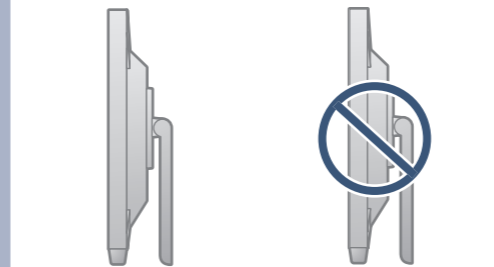
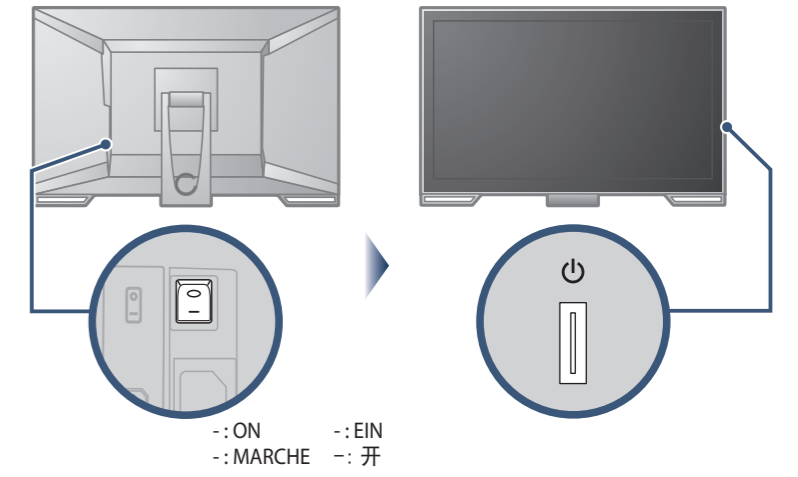
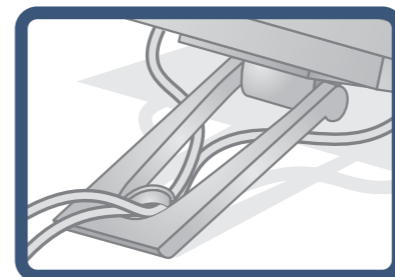
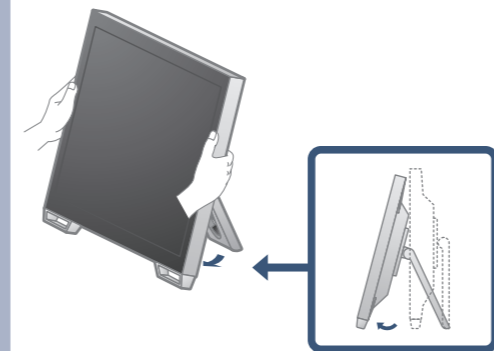
**No-Picture Problem** If no picture is displayed on the monitor even after the following remedial action is taken, contact your local EIZO representative.

Problem	Possible cause and remedy	Problem	Possible cause and remedy
Power indicator does not light.	<ul style="list-style-type: none"><li>Check whether the power cord is connected properly.</li> <li>Turn the main power switch on.</li> <li>Press <span>⏻</span>.</li> <li>Turn off the main power, and then turn it on again a few minutes later.</li></ul>	<b>No picture</b>	Power indicator is flashing orange.
Power indicator is lighting blue.	<ul style="list-style-type: none"><li>Increase "Brightness", "Contrast", or "Gain" in the Setting menu.</li></ul>		
<b>No picture</b>	<ul style="list-style-type: none"><li>Switch the input signal with <b>SIGNAL</b>.</li> <li>Move the mouse or press any key on the keyboard.</li> <li>Check whether the PC is turned on.</li> <li>Depending on the PC and graphics board, the input signal is not detected and the monitor does not recover from the power saving mode. If the screen is not displayed even after moving the mouse or pressing any key on the keyboard, turn the monitor off and on using its power button. When the screen is displayed, execute the following procedure. The problem may be improved. <ol style="list-style-type: none"><li>Press <span>⏻</span> to turn off the monitor.</li> <li>Press <span>⏻</span> holding <b>MODE</b> down to turn on the monitor.</li></ol> "x" is displayed on the menu title of "Information" of the Setting menu. <ol style="list-style-type: none"><li>Reboot the PC.</li></ol> To return to the previous setting, perform steps 1 to 3 again.</li></ul>	<b>The message appears.</b>	The message shows that the input signal is out of the specified frequency range. Example: <div data-bbox="884 390 1056 453" data-label="Image"> </div>
Power indicator is lighting orange.			<ul style="list-style-type: none"><li>Check whether the PC is configured to meet the resolution and vertical scan frequency requirements of the monitor.</li> <li>Reboot the PC.</li> <li>Select an appropriate setting using the graphics board's utility. Refer to the User's Manual of the graphics board for details.</li></ul>

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## When Using as a Touch Panel Monitor

There are multiple touch panel setting methods as shown below. Configure settings to meet your usage applications. When setting is completed, execute the "TPOffset" application on the "EIZO LCD Utility Disk" (CD-ROM), and adjust the sensitivity of the touch panel.

	Windows standard driver (Windows 8.1 / Windows 7)	Touch panel driver supplied by EIZO	
		Touch digitizer mode (Touch.mfd*)	Mouse emulation mode (Mouse.mfd*)
Driver	You do not need to install the driver.	Install the touch panel driver on the "EIZO LCD Utility Disk" (CD-ROM).	
Setting	Use Windows Control Panel.	Use the "Touch Screen Properties" supplied with the touch panel driver.	
Operation	You can not set the touch sound.	You can set the touch sound.	
	Only one touch panel monitor can be connected.	Multiple touch panel monitors can be connected.	
	Up to five simultaneous touch points are available.	Up to five simultaneous touch points are available.	Only single point touch is available.
	Enables the same operations as those of the mouse as well as multi-touch operations such as image zoom and rotation.	Enables the same operations as those of the mouse as well as multi-touch operations such as image zoom and rotation.	The cursor reacts in the same way when you operate the mouse.

\*1 Select "Basic Setting" - "Configuration File" in "Touch Screen Properties", and select the desired configuration file. In Windows XP, only the mouse emulation mode is available. If you select the touch digitizer mode, the touch position may not be recognized correctly.

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